

# tropicalgin



**RIDE THE COLOURS !**



**Clinical**  
Chromatic Alginate

**Zhermack®**  
CLINICAL



# tropicalgin

Let the colour be your guide!

## EXCLUSIVE CHARACTERISTICS

ZHERMACK presents rapid setting chromatic dust free alginate with three distinct phases:

- Shocking Pink phase:  
*mixing time*



45" max

- Salmon Pink phase:  
*working time*



1'35"

From the start of mixing time

- Yellow phase:  
*insertion into the mouth*



2'35"

Setting time

- rapid working and setting times
- thixotropic consistency
- rigid after setting
- excellent elastic recovery
- reliable results

## INDICATIONS

TROPICALGIN is ideal for:

- removable prosthesis
- opposing models for fixed and removable prosthesis
- study models
- temporary crown & bridge impressions



# tropicalgin



## TROPICAL FLAVOUR

Patients find the product's *tropical flavour extremely pleasant.*

## TECHNICAL DATA

Mixing time	45" max
Working time*	1' 35"
Time in mouth	1'
Setting time	2' 35"
Elastic recovery	99%
Permanent deformation	11,5%
Compressive strength	1,4 MPa

\*From the start of mixing time

## A WIDE RANGE OF ACCESSORIES



Flexible rubber mixing bowl, measure sets and spatulas



Hi-Tray impression tray range and Algitrax detergent

Zhermack  
CLINICAL



## Packaging

C302240 TROPICALGIN	1 bag 453 g
C302242 TROPICALGIN INTRO KIT	2 x 453 g bags + measures + container
C302245 TROPICALGIN	1 single dose sachet, 18 g
C302070 HYDROGUM 5	1 bag 453 g
C302071 HYDROGUM 5 INTRO KIT	2 x 453 g bags + measures + container
C302075 HYDROGUM 5	1 single dose sachet, 21 g
C302025 HYDROGUM	1 bag 500 g
C302041 HYDROGUM INTRO KIT	2 x 500 g bags + measures + container
C302051 HYDROGUM	1 single dose sachet, 18 g
C302060 HYDROGUM SOFT	1 bag 453 g
C302044 HYDROGUM SOFT	1 single dose sachet, 18 g
C302205 NEOCOLLOID	1 bag 500 g
C302221 NEOCOLLOID INTRO KIT	2 x 500 g bags + measures + container
C302230 NEOCOLLOID	1 single dose sachet, 18 g
C302145 ORTHOPRINT	1 bag 500 g
C302161 ORTHOPRINT INTRO KIT	2 x 500 g bags + measures + container
C302171 ORTHOPRINT	1 single dose sachet, 18 g
C302086 PHASE PLUS	1 bag 453 g
C302111 PHASE PLUS	1 single dose sachet, 18 g

## Equipment

C303230	ALGHAMIX II Automatic alginate mixer (230 V)
C303215	ALGHAMIX II Automatic alginate mixer (115 V)
C303231	ALGHAMIX II Automatic alginate mixer (230 V) UK version

## Accessories

C300990	PLASTIC ALGINATE SPATULAS set of 6 spatulas
C300992	FLEXIBLE RUBBER MIXING BOWL
C300960	LONG LIFE BAG 100 pcs package
C400431	ALGITRAY LIQUID Detergent for removing alginate from the impression tray. 2 l bottle
C400430	ALGITRAY POWDER 1 kg tub + measure
C300900	1 set of measures for Hydrogum 5 (1 water + 1 powder)
C300910	1 set of measures for alginates (1 water + 1 powder)

## Accessories Equipment

XR0250600	350 cc Alginate bowl - light blue
XR0250601	350 cc Alginate bowl - orange
XR0250602	350 cc Alginate bowl - navy blue
XR0250603	350 cc Alginate bowl - green
XR0250604	350 cc Alginate bowl - white
XR0250606	350 cc Alginate bowl - grey
XR0250650	600 cc Gypsum bowl - grey
C300990	Fluo spatula (x6)
C303030	Mixing block (20 sheets)
XR0250670	Silicone mixing disk
C207200	Silicone mixing spatula

## BRANCH

**GERMANY**  
Zhermack GmbH  
Deutschland  
Öhlmühle 10 - D-49448 Marl  
Tel. +49 - 0 54 43 / 20 33 - 0  
Fax +49 - 0 54 43 / 20 33 - 11  
info@zhermack.de  
www.zhermack.com

**POLAND**  
Zhermapol  
ul. Augustówka 14  
02 - 981 Warszawa - Polska  
Tel. +48 - 22 858 82 72  
+48 - 858 73 41  
Fax +48 - 22 642 07 14  
biuro@zhermapol.pl  
www.zhermack.com

**USA**  
Zhermack Inc.  
P.O. Box 4195, River Edge  
New Jersey 07661-4195  
Tel. +1 (877) 819-6206  
+1 (732) 389 8540  
Fax +1 (732) 389 8543  
info@zhermackusa.com  
www.zhermackusa.com

## COUNTRY CONTACTS

**UK AND IRELAND**  
+44 - (0) 7870 690811  
uk@zhermack.com

**SPAIN**  
Tel. 900 99 39 52  
info.spain@zhermack.com

**FRANCE**  
Tel. 0800 - 915083  
info.france@zhermack.com

**RUSSIA**  
Tel. +7 916 930 1191  
info.russia@zhermack.com

**MEXICO AND CENTRAL AMERICA**  
Tel. +52 - 1 - 55 - 91980497  
info.mexico@zhermack.com

**SOUTH AMERICA**  
Tel. +57 - 310 - 2601777  
info.colombia@zhermack.com

## Zhermack SpA

Via Bovazecchino, 100 - 45021 Badia Polesine (RO) ITALY  
Tel. +39 - 0425 597611 Fax +39 - 0425 53596 info@zhermack.com www.zhermack.com