

SYSTEM

zetaplus



A MATTER OF TRUST

Clinical

C-Silicone Impression Material

Zhermack®

CLINICAL




zetaplus system

Solutions for all clinical situations.

ZHERMACK reaffirms the success of the ZETAPLUS SYSTEM.

The ZETAPLUS SYSTEM, has been appreciated worldwide for 25 years, has maintained its high performance and constant reliability in order to obtain the best possible clinical results.

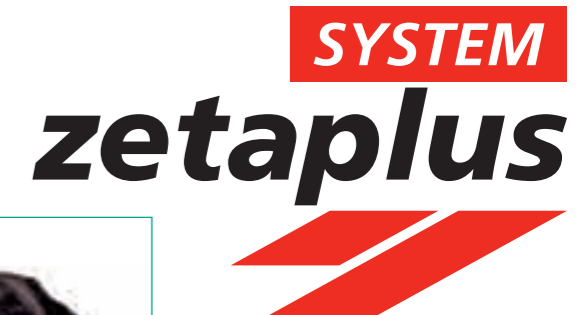

REDUCED MARGIN OF ERROR

› *Clear colour contrast to reduce the risk of error*, consequently saving time and material.

BALANCED REACTIVITY

- › *Ideal balance* between working time and setting time. (*Snap-set* effect).
- › Optimal dosage of the catalyst increases the available working time and reduces contraction.

“FLOW CONDITIONING” EFFECT

› Increased *flowability and extreme dimensional stability* are achieved thanks to SAS (Synthetic Amorphous Silicas) high-activity spheroid fillers.

zetaplus system

“Flow conditioning” effect for high-detail impressions.

ZETAPLUS with ORANWASH L and THIXOFLEX M

ZETAPLUS, soft to mix with rigid set

The *“flow conditioning” effect* helps the material *flow* and *reduces pressure* on the soft tissue. Due to its rigid set, **ZETAPLUS** is the recommended *putty* for the two-stage technique.

ORANWASH L, hydrophilic and fluid

This light bodied hydrophilic flowable silicone is suitable for the two-stage technique.

THIXOFLEX M, thixotropic, flowing and hydrophilic

This *medium bodied thixotropic silicone guarantees accurate detail* in humid environments thanks to its *intense hydrophilic character. Recommended for the single-stage technique.*



Soft to mix due to the “flow conditioning” effect



Flowability in the sulcus

ZETAPLUS SOFT with ORANWASH VL

ZETAPLUS SOFT, flowability and elasticity


This *soft to mix*, with *elastic setting putty* is suitable for any clinical situation and is particularly recommended for the *single-stage* technique.



Optimal rigidity

ORANWASH VL, hydrophilic and very fluid

This *very light bodied hydrophilic silicone is suitable for all impression techniques.*

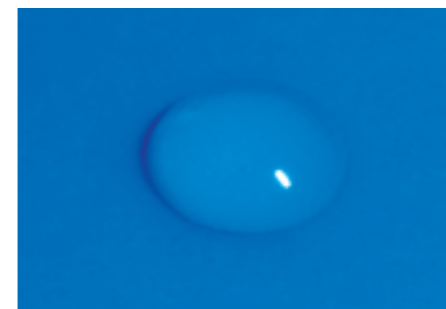

Hydrophilic after 1 minute




Flowability while working

indurent gel and zetaplus system

The ideal colour contrast.

The **INDURENT GEL** catalyst guarantees a *homogeneous mix* for use with the putties and the washes of the **ZETAPLUS SYSTEM**.

Ideal balance between working time and setting time

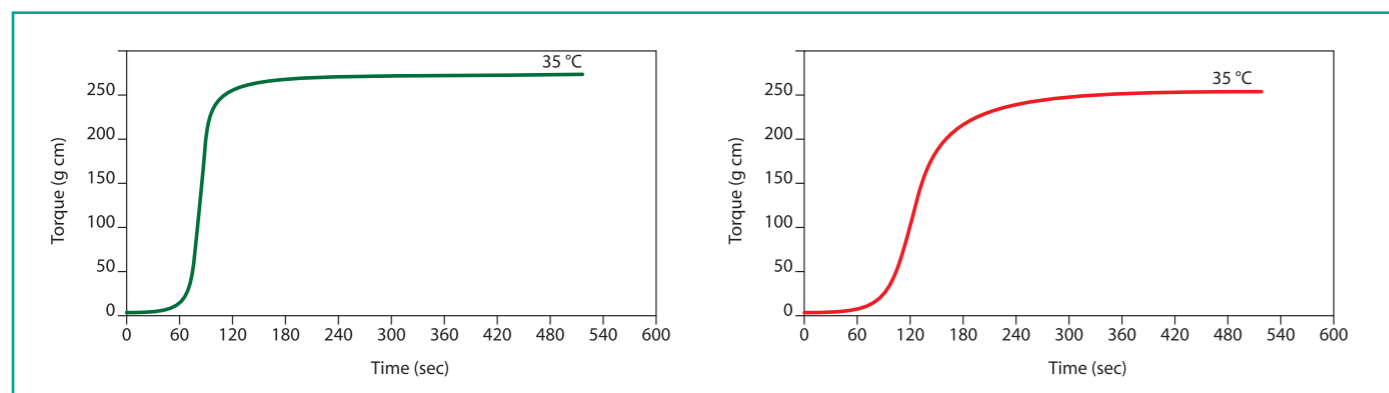

SNAP-SET effect (*zetaplus - zetaplus soft*)

SNAP-SET effect (*oranwash VL – oranwash L*)

TECHNICAL INFORMATION	ZETAPLUS	ZETAPLUS SOFT	ORANWASH VL	ORANWASH L	THIXOFLEX M
Mixing time	30 sec.	30 sec.	30 sec.	30 sec.	30 sec.
Total working time	1 min. 15 sec.	1 min. 15 sec.	1 min. 30 sec.	1 min. 30 sec.	2 min.
Time in Mouth	3 min. 15 sec.	3 min. 15 sec.	3 min. 30 sec.	3 min. 30 sec.	3 min. 30 sec.
Setting time	4 min. 30 sec.	4 min. 30 sec.	5 min.	5 min.	5 min. 30 sec.
Deformation under pressure (min/max)	2% - 5%	2% - 5%	2% - 8%	2% - 8%	2% - 8%
Elastic recovery	> 98%	> 98%	> 99%	> 99%	> 99%
Dimensional stability	< -0.2%	< -0.2%	< -0.7%	< -0.7%	< -0.7%

Performance of ZETAPLUS SYSTEM

zetaplus system
Reliable in any situation.

SYSTEM**zetaplus**

GPLCOMMUNICATION.COM

F121019/C - 09/2010

Packaging

C100600 ZETAPLUS	1 tub 900 ml (1.53 Kg) + scoop
C100312 ZETAPLUS	1 tub 1800 ml (3 Kg) + scoop
C100468 ZETAPLUS	1 container, 10 Kg + scoop
C100469 ZETAPLUS	1 container, 25 Kg
C100730 ZETAPLUS INTRO KIT L	1 tub Zetaplus 900 ml (1.53 Kg) + scoop + 1 tube Oranwash L 140 ml + 1 tube Indurent gel 60 ml + mixing pad
C100720 ZETAPLUS TRIAL KIT L	1 tub Zetaplus 200 ml + scoop + 1 tube Oranwash L 40 ml + 1 tube Indurent gel 60 ml + mixing pad
C100610 ZETAPLUS SOFT	1 tub 900 ml (1.53 Kg) + scoop
C100612 ZETAPLUS SOFT	1 tub 1800 ml (3 Kg) + scoop
C100613 ZETAPLUS SOFT	1 container, 10 Kg + scoop
C100614 ZETAPLUS SOFT	1 container, 25 Kg
C100740 ZETAPLUS SOFT INTRO KIT VL	1 tub Zetaplus Soft 900 ml (1.53 Kg) + scoop + 1 tube Oranwash VL 140 ml + 1 tube Indurent gel 60 ml + mixing pad
C100710 ZETAPLUS SOFT TRIAL KIT VL	1 tub Zetaplus Soft 200 ml + scoop + 1 tube Oranwash VL 40 ml + 1 tube Indurent gel 60 ml + mixing pad
C100650 ORANWASH VL	1 tube 140 ml
C100660 ORANWASH L	1 tube 140 ml
C100670 THIXOFLEX M	1 tube 140 ml

Catalyst

C100700 INDURENT GEL	1 tube 60 ml
C100390 INDURENT LIQUID	1 bottle 30 ml

Equipment

C303230 ALGHAMIX II	Mixing machine for alginate and silicone (230 V)*
C303215 ALGHAMIX II	Mixing machine for alginate and silicone (115 V)*

Accessories

U113250	Stainless steel spatula for silicones - sterilizable
D510010	Putty Cut
C207201	Mixing pad, 20 sheets
XR0250670	Mixing disk for silicones for Alghamix II
C303030	Mixing pad for Alghamix II, 20 sheets

*Disk for silicone available as an accessory

BRANCH

GERMANY
Zhermack GmbH
Deutschland
Öhlmühle 10 - D-49448 Marl
Tel. +49 - 0 54 43 / 20 33 - 0
Fax +49 - 0 54 43 / 20 33 - 11
info@zhermack.de
www.zhermack.com

POLAND
Zhermapol Sp. z o.o.
ul. Augustówka 14
02 - 981 Warszawa - Polska
Tel. +48 - 22 858 73 11
+48 - 22 858 73 41
Fax +48 - 22 642 07 14
biuro@zhermapol.pl
www.zhermack.pl

USA
Zhermack Inc.
P.O. Box 4195, River Edge
New Jersey 07661-4195
Tel. +1 (877) 819-6206
+1 (732) 389 8540
Fax +1 (732) 389 8543
info@zhermackusa.com
www.zhermackusa.com

COUNTRY CONTACTS

UK AND IRELAND
Tel. +44 - 07870 690811
uk@zhermack.com

SPAIN
Tel. 900 99 39 52
info.spain@zhermack.com

FRANCE
Tel. 0800 - 915083
info.france@zhermack.com

RUSSIA
Tel. +7 916 930 1191
info.russia@zhermack.com

MEXICO AND CENTRAL AMERICA
Tel. +52 - 1 - 55 - 91980497
info.mexico@zhermack.com

SOUTH AMERICA
Tel. +57 - 310 - 2601777
info.colombia@zhermack.com

Zhermack SpA

Via Bovazecchino, 100 - 45021 Badia Polesine (RO) ITALY
Tel. +39 - 0425 597611 Fax +39 - 0425 53596 info@zhermack.com www.zhermack.com

Zhermack®
BEYOND INNOVATION

